



USB Digital Endoscope

Model No. Y003 User Guide

Introduction

Thank you for purchasing Supereyes portable Digital Endoscope. Y003 handheld digital endoscope is compatible with both computer and smart phone. It's easy and simple to use for family health care. Prior to the use, we recommend reading this manual to get the best out of this item.

Specifications

- Resolution: 0.3 MP
- Frame rate: Max 30 FPS
- Adjustable LED illumination
- Interface: USB 2.0

Items in this package


- Y003 itself
- Disk
- This user guide

System Requirement

- Windows XP 7, 8, 10 or above, Mac OS
- Interface: USB 2.0
- HDD storage space: 600MB and above

Installation and use

1. Plug the USB interface into computer or smartphone;
2. Rotate the manual focus ring at the end of the endoscope till get clear image;
3. Adjust the LED illumination brightness by control wheel in the wire;
4. Press the snap shot button in the wire to take photo.

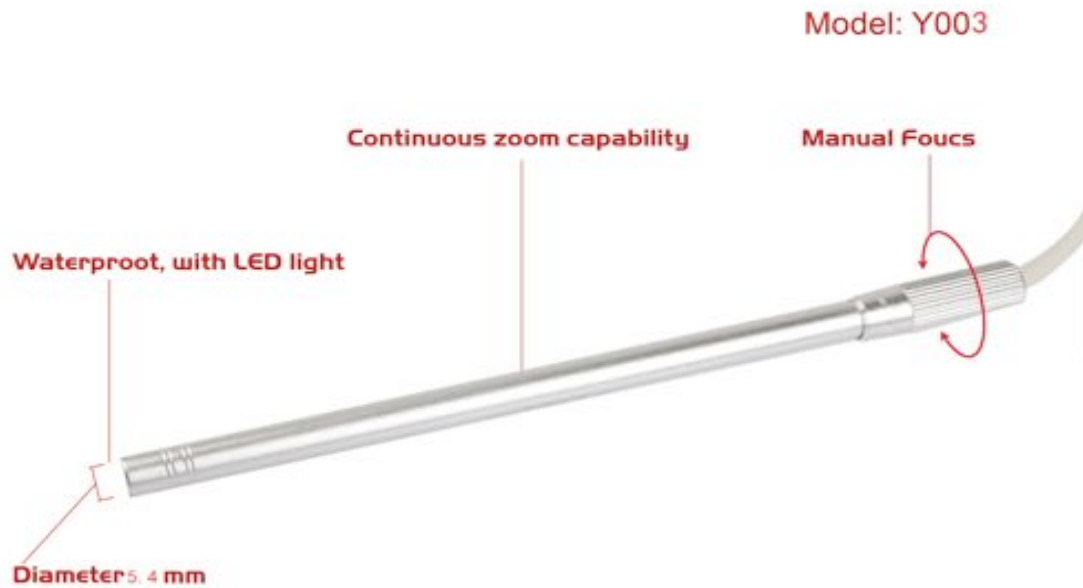

Trouble Shooting

	Trouble vision	Possible Reason	Solution
1	Blank screen	LED is closed; Preview is closed; No suitable devices; Incorrect connection with PC.	Open LED and rotate the lens; Open preview; Choose devices; Connect USB on the mainboard.
2	Vague picture, discoloration, slow speed	Incorrect focus; Incorrect connection with PC; Resolution is too high; The Configuration of PC is too low;	Adjust focus; Connect mainboard USB; Select low resolution; Choose new PC;
3	Picture distortion	Incorrect connection with PC; Defective endoscope.	Connect mainboard USB; Contact to repair.
4	Cannot recognized	Incorrect connection with PC; Trouble inside of endoscope.	Connect mainboard USB;; Contact to repair.

Precautions

- Please do not touch lens and other optics, otherwise, product performance may be influenced;
- Please do not disassemble product to avoid danger and product damage;
- Please use and store it under this environment, temperature: 0°C ~ 40°C; humidity: 45% ~ 85%
- In case of dropping tiny object or liquid into microscope, please disconnect the USB cable immediately and send product to maintenance center for quality inspection and repair

Certificates



SHENZHEN SUPEREYES CO., LTD

3F, Bldg1, Zone B, Hekan Industry Area, Wuhe South Rd, Bantian, Loanggang Dist, Shenzhen, China, 518129

Customer service: help@supereyes.cc